


# Animal Allergy

## The safe test for your friend.

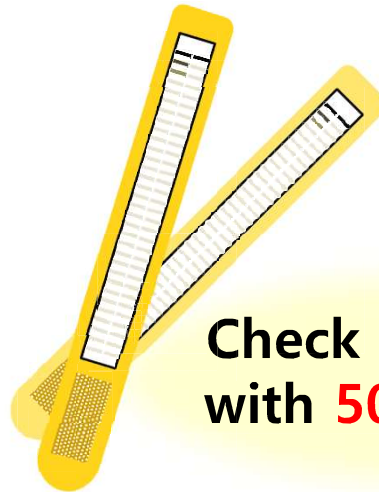


Check **66 allergens**  
with **50ul** of serum

### MULTIPLE ALLERGEN SIMULTANEOUS TEST

#### What is special about ANITIA *Canine IgE* ?

- Small blood volume for test
  - Requires only **50 uL** of serum for **66 allergens** for the test
- Screening and semi-quantitative detection of allergen specific IgE antibodies in Canine serum
- Applicable with Automatic & Semi-Automatic Instruments
- Line blot immunoassay: **Reproducible principle** compare to conventional test kit
- On-strip calibration

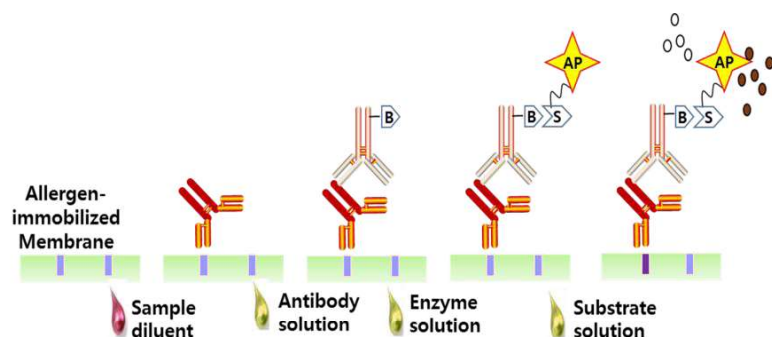

: sIgE in Sample

: Anti-canine-IgE

B: Biotin S: Streptavidin AP: Alkaline Phosphatase  
O: Substrate : Colored product

## 66 Allergens on panel

Group	No.	Allergen
MITE	1	<i>Dermatophagoides pteronyssinus</i>
	2	<i>Dermatophagoides farinae</i>
	3	<i>Acarus siro</i>
	4	<i>Tyrophagus putrescentiae</i>
TREE	5	Alder/ Birch
	6	Hazel
	7	Maple leaf sycamore
	8	Willow/ Cottonwood
	9	Oak
	10	White pine
	11	Acacia
	12	White ash
WEED	13	Common ragweed
	14	Plantain
	15	Mugwort
	16	Sheep's sorrel
GRASS	17	Bermuda grass
	18	Orchard/ Timothy grass
	19	Ryegrass
	20	Cultivated rye
ANIMAL	21	Cat epithelium & dander
	22	Wool, sheep
	23	Feather mix
MOLD	24	<i>Penicillium notatum</i>
	25	<i>Cladosporium herbarum</i>
	26	<i>Aspergillus fumigatus</i>
	27	<i>Candida albicans</i>
	28	<i>Alternaria alternata</i>
INSECT	29	<i>Malassezia pachydermatis</i>
	30	Flea

No.	Allergen	Group
31	Cockroach	INSECT
32	Pork	MEAT
33	Beef	
34	Duck meat	
35	Chicken	
36	Lamb meat	
37	Turkey meat	GRAIN
38	Red deer	
39	Wheat	
40	Corn	
41	Rice	
42	Pea	VEGETABLE, FRUIT
43	Soy bean	
44	Carrot	
45	Potato	
46	Sweet potato	
47	Pumpkin	DAIRY
48	Tomato	
49	Apple	
50	Yeast, baker's	
51	Egg white	
52	Egg yolk	SEAFOOD
53	Milk	
54	Cheddar/ Gouda cheese	
55	Crab/ Shrimp	
56	Codfish	
57	Tuna	NUT
58	Salmon	
59	Mackerel	
60	Peanut	



### ANITIA *Canine IgE*

- Allergen panel
- Sample diluent
- Antibody solution
- Enzyme solution
- Substrate solution
- Washing solution 20x



[Storage condition] Store at 2~8°C

[Shelf life] 18 months after date of manufacture, 3 months after open

[Packaging unit] 20 Tests/ Kit